


LOVI Silicone Nipple Protectors. Available in two sizes: S – Small, M/L – Standard. Made of soft silicone and designed for use when nipples are cracked or sore, these nipple shields help nipples to heal so that you can continue to breastfeeding. The shape allows your baby to feel and smell your skin and they will return feeding directly from your nipples once they have healed. The silicone is tasteless and odorless and does not affect the taste of the milk. They allow your baby to feel comfortable and natural during breastfeeding. Warning! Inspect carefully before each use. Throw away at the first signs of damage or weakness. Keep the product clean. Before the first and each subsequent use sterilize by means of boiling water for at least 3 minutes. Remember to apply the protector on a dry and clean breast. Do not use any cream or lubricant. If you are in any doubt about using this product speak to your Health Professional for advice. Warning! This product is not a toy. Keep out of reach from children when not in use. Retain these instructions for later use. Batch code and expiry date are on the packaging.


1. Stimulate the nipple slightly using your fingers so that it stands up.
2. Place the centre of the nipple protector as far onto the nipple as possible.
3. Pull the shield against the breast and smooth onto the skin making sure that the cut away section is towards the top so that it allows your baby's nose to touch your skin during feeding.
4. If necessary gently hold the shield in place and you can begin feeding.

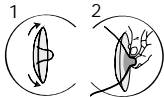

k001



GB

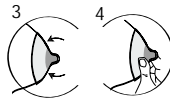

4

